

## INTRODUCTION

The properties of sleep architecture are an under-evaluated but highly relevant aspect for children with epilepsy [1,2]. Conventional recording methods for assessing sleep, which require EEG, EMG and EOG electrodes, are quite cumbersome and limit the examinations to single-night recordings. A more discreet and practical setup would open the path towards easier and longer-term recordings, additionally allowing to investigate daytime naps and multi-night correlations [3]. **The current study explores the feasibility of adopting a behind-the-ear EEG montage for automated sleep staging in a pediatric population.**

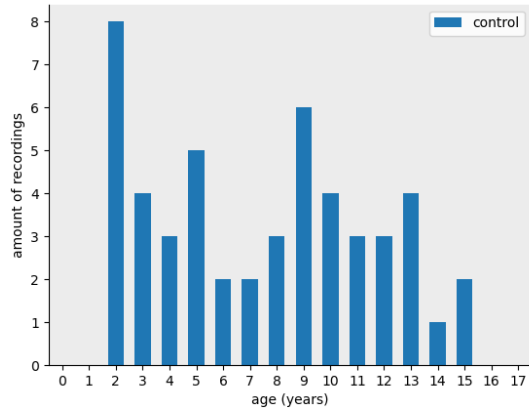


Figure 1. Behind-the-ear EEG was simulated in 50 pediatric patients from 10-20 EEG. Left panel: Example of a behind-the-ear EEG setup in a pediatric subject. Right panel: The histogram of the pediatric patients included in the study.

## METHODS

- **50 overnight-recordings with conventional 10-20 set-up from pediatric patients** (without epileptic abnormalities during the recorded period) were selected from a dataset from Saint-Luc University Hospital (Brussels, Belgium).
- 3 independent experts manually scored sleep staging (Wake, N1, N2, N3, REM).
- **Behind-the-ear montage was simulated** from 9 additional recordings which had simultaneous 10-20 and behind-the-ear EEG (provided by Byteflies). For each recording, the ordinary least-squares solution (OLS) was obtained. The average OLS solution was used to simulate a behind-the-ear montage for the 50 pediatric patients.
- **Sleep staging models were trained** for automated sleep scoring (Wake, NREM, REM) for the 10-20 set-up as well as the behind-ear-simulation. This algorithm included the extraction of several features and a gradient boosting model for classification.
- **Performance evaluation** was based on Cohen's kappa coefficient ( $\kappa$ ) using a 5-fold cross-validation strategy. The inter-expert agreement was used as baseline performance.
- **Statistical comparisons** was done with the two-tailed Wilcoxon signed-rank test and a significance threshold of 0.01.

## RESULTS

### Simulating behind-the-ear EEG from 10-20 EEG

The average OLS solution to simulate the behind-the-ear from full scalp EEG montage uses a bipolar reference of channels T3 and T4. The amplitude of the bipolar signal is furthermore reduced by 70% to match the reference behind-the-ear signal.



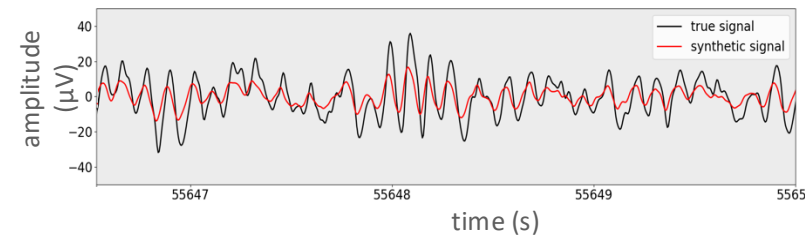


Figure 2. Left panel: Average OLS solution to simulate behind-the-ear montage from 10-20 EEG. Color indicates the weights that were assigned to each channel. Right panel: Overlay of true and synthetic behind-the-ear signal shows a good match.

### Automated sleep staging

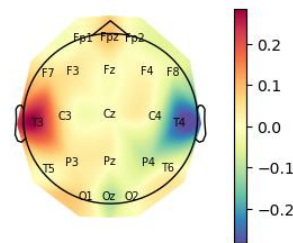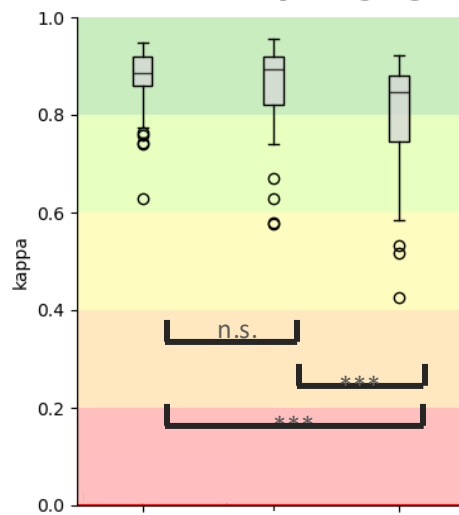

The average agreement among the three experts for full-montage recordings was high ( $\kappa = 0.87 \pm 0.06$ ). Note that this is higher than in literature due to the many Wake stages (~50%, 24-hour recordings) and the grouping of N1, N2 and N3 stages in one NREM group. The model that uses the **full-scalp EEG** achieves near-perfect agreement with the consensus scoring ( $\kappa = 0.87 \pm 0.09$ ), which was not significantly different compared to the interrater agreement ( $p = 0.74$ ). From **behind-the-ear EEG**, the model reached a slightly lower, albeit still high, agreement compared to consensus scoring ( $\kappa = 0.81 \pm 0.1$ ). This is significantly different from the full-scalp model and the interrater agreement (both  $p < 0.001$ ).

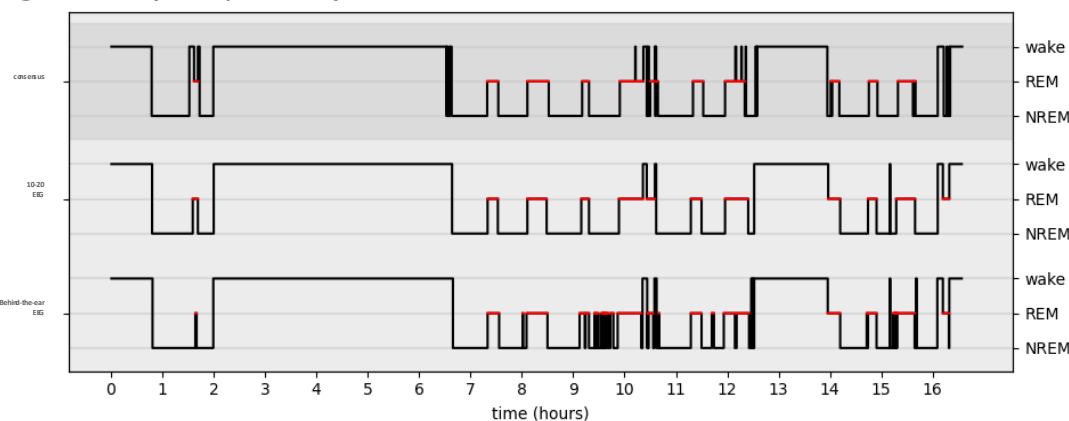


Figure 4. Example of similarity between hypnograms obtained by the expert consensus scoring the standard scoring based on the 10-20 EEG and the automated scoring based on the simulated behind-the-ear EEG.

## CONCLUSION

- Sleep staging can be reliably assessed automatically from a behind-the-ear EEG setup.
- The algorithm reliably detects NREM stages and wakefulness, detection of REM stage is less reliable, likely due to limited spatial sampling.
- This holds promise toward a broader clinical utility and ease-of-use of long-term sleep assessment (incl. its use in ambulatory settings) combining non-intrusive wearable hardware with automated analysis.

## LIMITATIONS

- No assessment on actual behind-the-ear recordings.
- Cross-validation within one datasets does not allow to assess generalizability across population and

## ACKNOWLEDGEMENTS

This study was funded by UCB Inc.

## References

- [1] Stores, G., Wiggs, L., Campling, G. (1998). Sleep disorders and their relationship to psychological disturbance in children with epilepsy. *Child: care, health and development*, 24(1), 5-19.
- [2] Manni, R., & Terzaghi, M. (2010). Comorbidity between epilepsy and sleep disorders. *Epilepsy research*, 90(3), 171-177.
- [3] Libenson, M. H., Haldar, A., & Pinto, A. L. (2014). The stability of spike counts in children with interictal epileptiform activity. *Seizure*, 23(6), 454-456.